



# HOW DO WE CLEAN OUR MANIKINS

1

## LUNG BAGS

We dispose of lung bags after each training session.

2

## HEAD REMOVED AND CLEANED

We Scrub the outer sevice using warm soapy water and brush clean, rise through with clean water.

3

## DISINFECT HEAD

We submerge the head in a chlorine solution - 60ml of household bleach to 4 liters of water for a min of 10 minutes.

4

## RINSE

We rinse all components in detergent-free water at 30 - 40 °C and towel dry the heads.

5

## AIR DRY

We dry the components throughly in a clean air enviroment



## SOAP SOLUTION

INLINE WITH MANUFACTURES GUIDELINES